


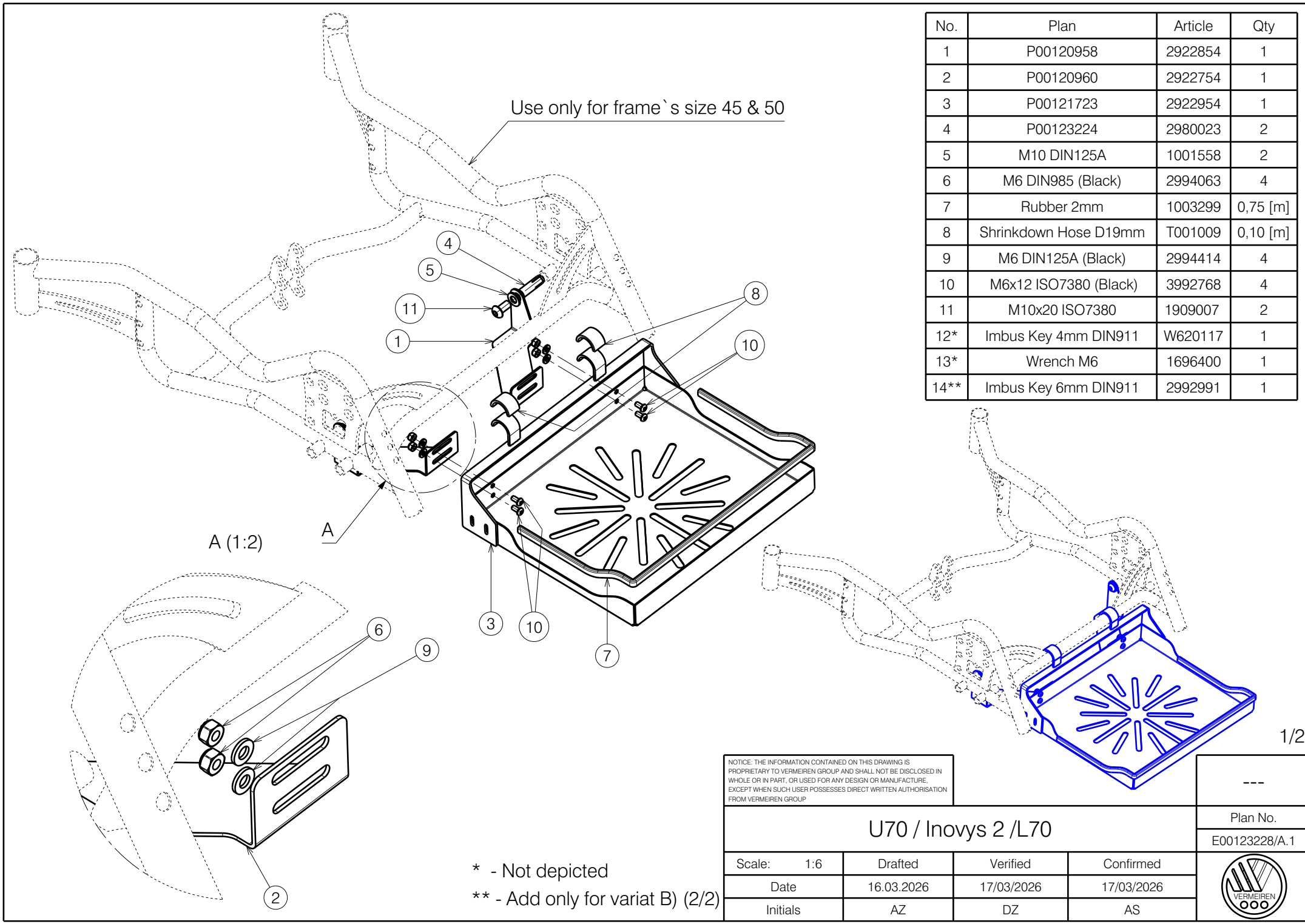

No.	Plan	Article	Qty
1	P00120958	2922854	1
2	P00120960	2922754	1
3	P00121723	2922954	1
4	P00123224	2980023	2
5	M10 DIN125A	1001558	2
6	M6 DIN985 (Black)	2994063	4
7	Rubber 2mm	1003299	0,75 [m]
8	Shrinkdown Hose D19mm	T001009	0,10 [m]
9	M6 DIN125A (Black)	2994414	4
10	M6x12 ISO7380 (Black)	3992768	4
11	M10x20 ISO7380	1909007	2
12*	Imbus Key 4mm DIN911	W620117	1
13*	Wrench M6	1696400	1
14**	Imbus Key 6mm DIN911	2992991	1


NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

U70 / Inovys 2 / L70				---
				Plan No.
				E00123228/A.1
Scale: 1:6	Drafted	Verified	Confirmed	
Date	16.03.2026	17/03/2026	17/03/2026	
Initials	AZ	DZ	AS	


* - Not depicted
** - Add only for variat B) (2/2)

WHEELS CONFIGURATION

T30

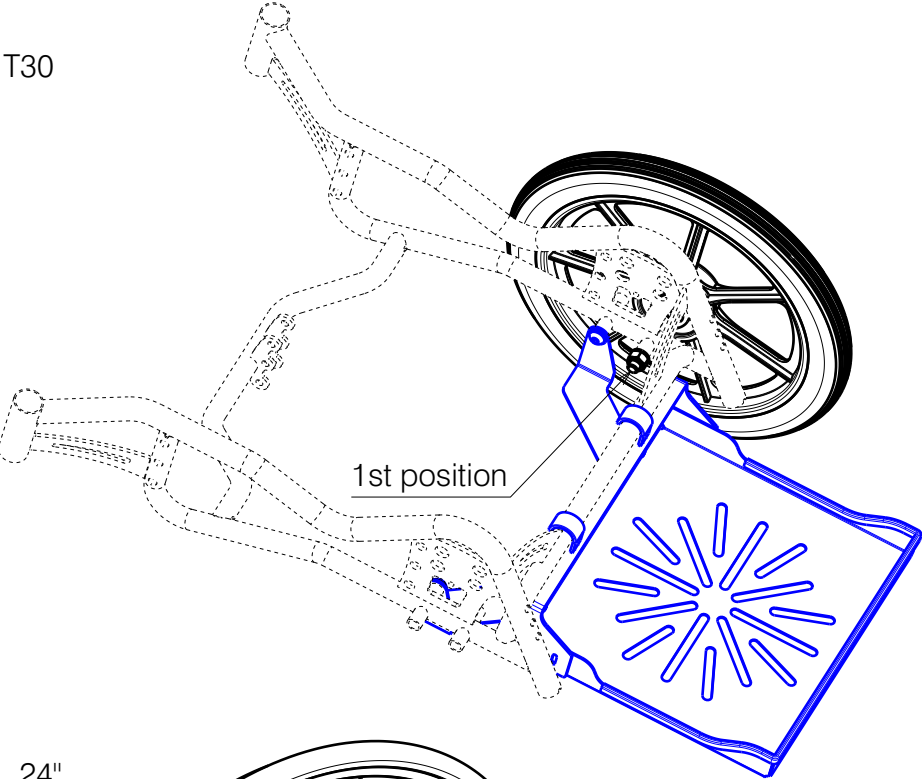


24"

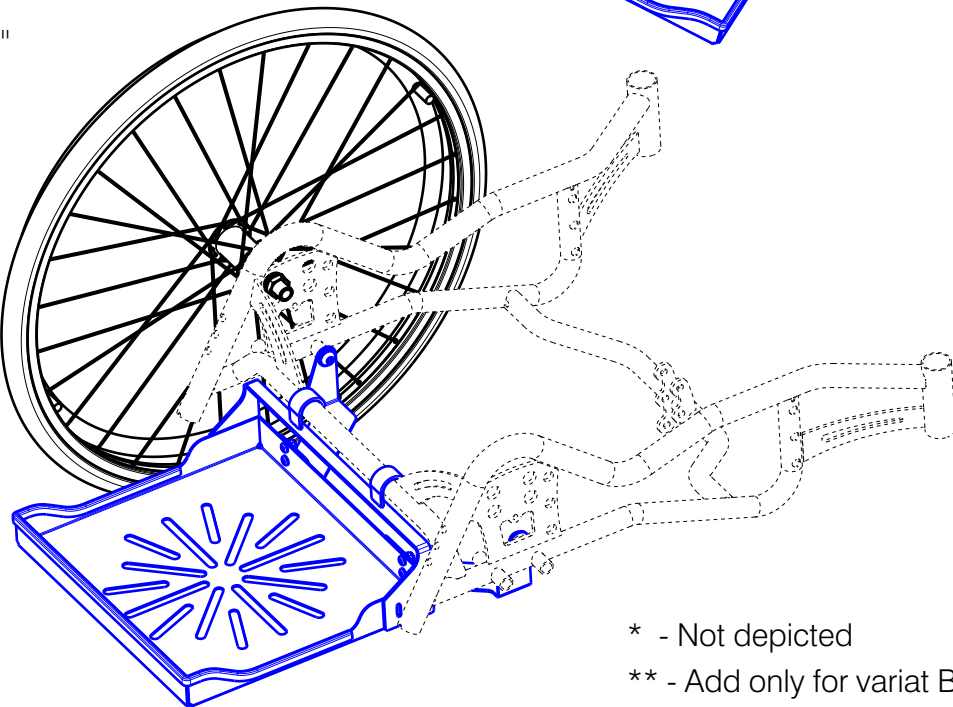

* - Not depicted
** - Add only for variat B) (2/2)

T30

- use without no.4, 5, 11.

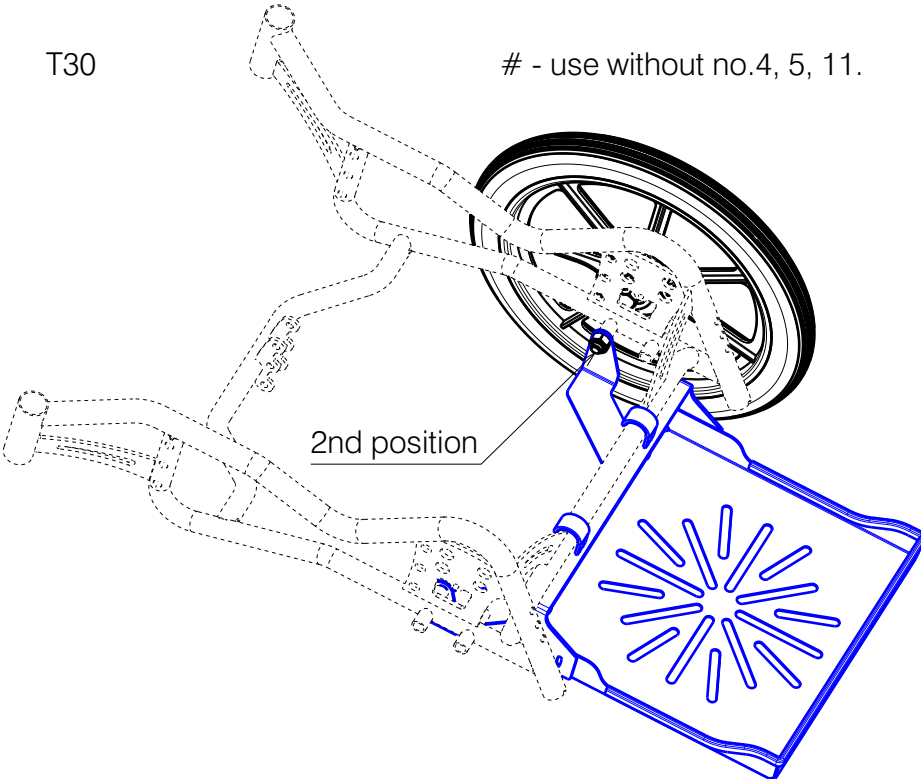

Assembly and packaging information

- A) Order Wheelchair with U70 option:
- 1) Part 3 (assembled with 8 & 7) pack separately in main box with wheelchair. Use foam and foil to protect.
 - 2) Add string bag U000123 with parts 6, 9, 10, 12*, 13* inside to main box with wheelchair.
 - 3) Parts assembled to wheelchair: 1,2, optionally 4,5,11.
- B) Order option U70 separately:
- 1) Pack all parts 1-14** using box: U000BZ7 / protect by foam and foil.

2/2

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

U70 / Inovys 2 /L70				Plan No. E00123228/A.1
Scale: 1:8	Drafted	Verified	Confirmed	
Date	16.03.2026	17/03/2026	17/03/2026	
Initials	AZ	DZ	AS	