


REMOVABLE SIDE GUARD NOT POSSIBLE WITH CAMBER 6°
 WITH OFFROAD WHEELS NOT POSSIBLE 4° AND 6°

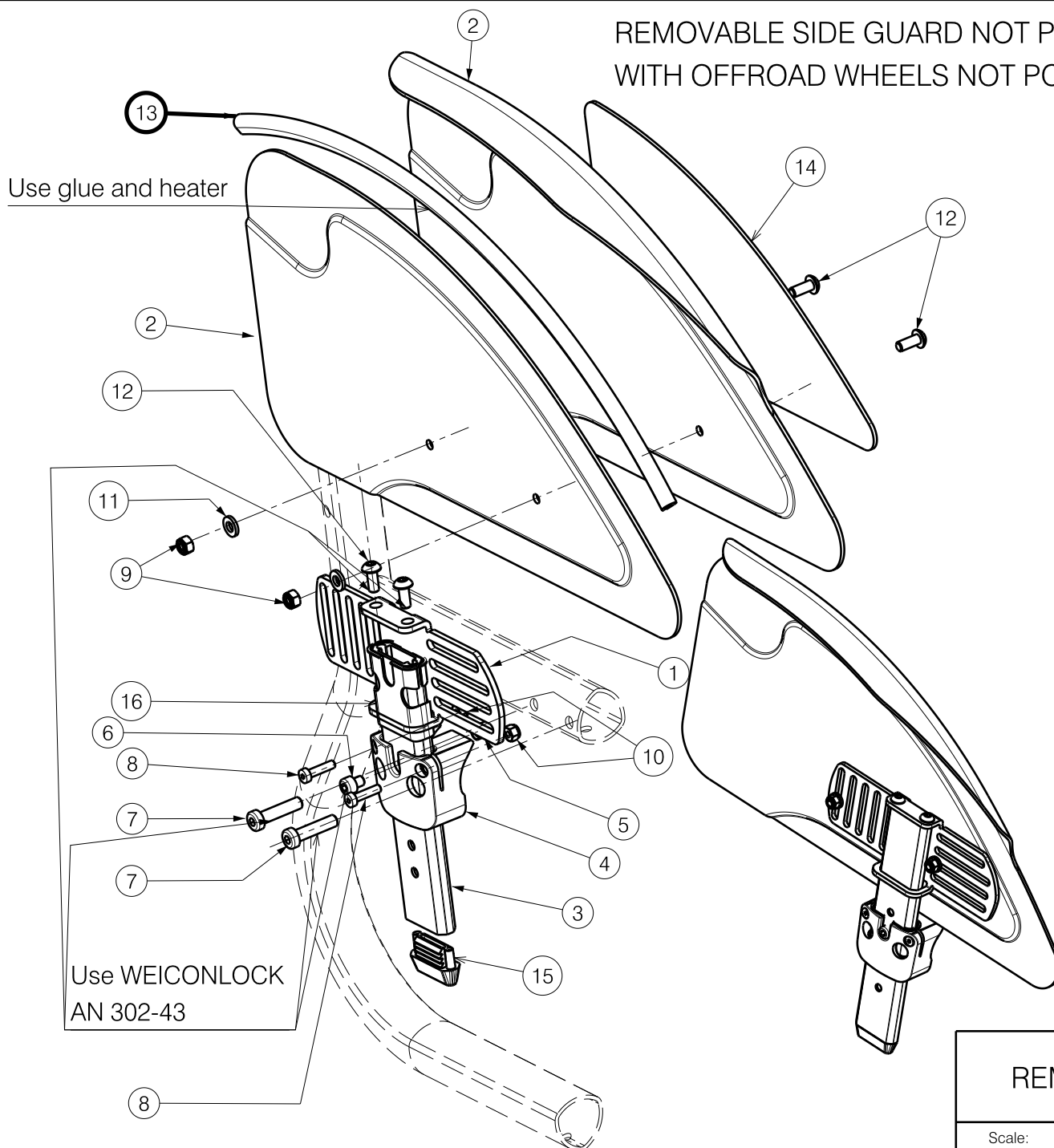

Material			1
No.	Plan	Article	Quantity
1	YP05755(R)	2723554	1
	YP05757(L)	2723654	1
2a	YP07443(R)	27635..	1
	YP07444(L)	27636..	1
2b	YP07999(R)low	26176..	1
	YP08000(L)low	26177..	1
2c	YP07446(R)	27637..	1
	YP07447(L)	27638..	1
2d	YP08487(R)low	26178..	1
	YP08488(L)low	26179..	1
2e	XP01938(R)carbon	2993612	1
	XP01941(L)carbon	2993615	1
2f	XP01939(R)carbon	2993613	1
	XP01973(L)carbon	2993631	1


Degrease the screws before using the glue.

Use WEICONLOCK
 AN 302-43


EP00248 / G

REMOVABLE SIDE GUARD TRIGO 36-50				Plan No. EP00248/ G
Scale: 1:3	Drafted	Verified	Confirmed	
Date	20/10/2017	18/02/2021	18/02/2021	
Initials	AS	MK	TK	


2a (L)

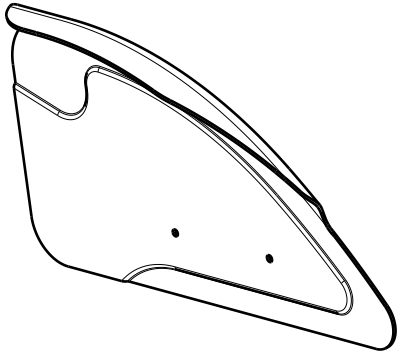


2b (S)

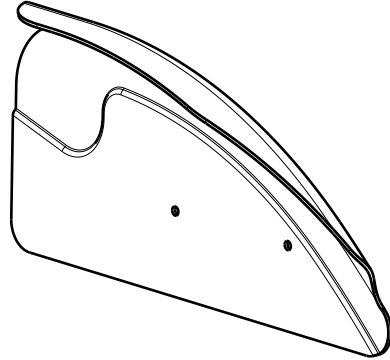


2c (L)

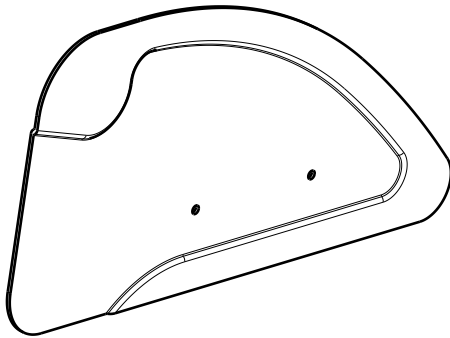


2d (S)

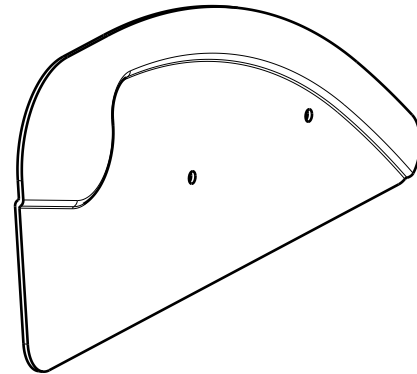

2e-carbon

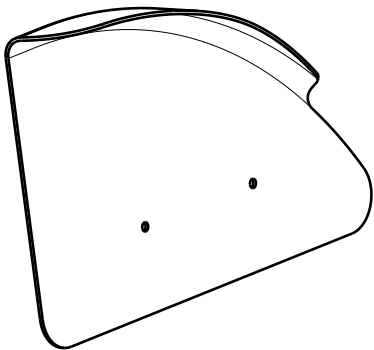

2f-carbon

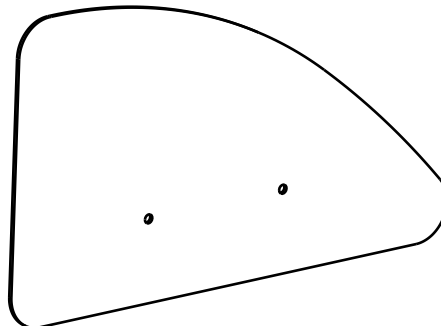

EP00248 / G

Material			
No.	Plan	Article	Quantity
3	XP02068	GP15099	1
4	XP03056	GP23299	1
5	XP01730	GP11200	1
6	M5X6_DIN912	1997822	1
7	M6x25	2994897	2
8	M4x16	2994070	2
9	Nut_M5_black	2994006	2
10	Nut_M4	1001678	2
11	M5_DIN_125_A	2994069	4
12	M5x12	2994074	4
13	---	1003299	0,8m
* 14	XP02274(R) (Large)	2996467	1
	XP02275(L) (Large)	2996468	1
	XP02272(R) (Small)	2996465	1
	XP02273(L) (Small)	2996466	1
15	XP03080	2996656	1
16	XP03091	2996662	1

*Dont use with carbon.

REMOVABLE SIDE GUARD TRIGO 36-50

Plan No.

EP00248/ G

Scale: 1:3

Drafted

Verified

Confirmed

Date

20/10/2017

18/02/2021

18/02/2021


Initials

AS

MK

TK



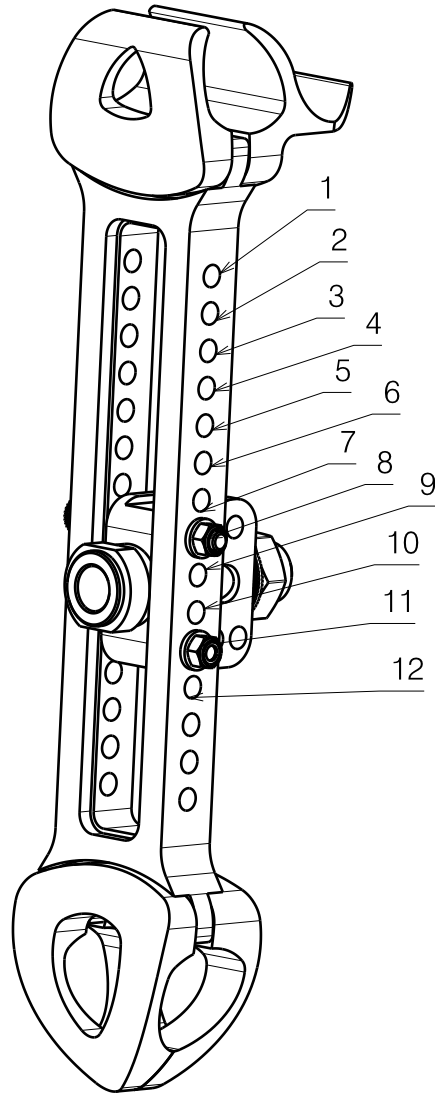

SEAT HEIGHT REAR 36-50																						
	315	325	335	345	355	365	375	385	395	405	415	425	435	445	455	465	475	485	495	505	515	525
18"		3	4	5	6	7	8	9	10	11	12											
20"			1	2	3	4	5	6	7	8	9	10	11	12								
22"					1	2	3	4	5	6	7	8	9	10	11	12						
24"								1	2	3	4	5	6	7	8	9	10	11	12			
26"											1	2	3	4	5	6	7	8	9	10	11	12

SMALL	26176../26177..	not possible 15mm
	26178../ 26179..	
LARGE	27635../27636..	not possible 15mm
	27637../ 27638..	

EP00248 / G

REMOVABLE SIDE GUARD TRIGO 36-50				Plan No.
				EP00248/ G
Scale: 1:3	Drafted	Verified	Confirmed	
Date	20/10/2017	18/02/2021	18/02/2021	
Initials	AS	MK	TK	