


no	part nu	part	QT		
1	1920656	Main frame ONE	1		
2	1918862	front fork	1		
3	1918856	STEERING AXLE ONE	1		

4	1920339	FRONTAXLE L	1		
5	1918854	FRONTAXLE R	1		
6					
7	1920443	COLOR PANEL SET FRONT green			
7	1920444	COLOR PANEL SET FRONTorange			
7	1920445	COLOR PANEL SET FRONTbleu			
7	1920446	COLOR PANEL SET FRONT pink			
8					
8					
8					
8					
9	1918852	FRONTCOVER	1		
10					
11					
12	1918851	SHROUD ONE	1		
13					
14					
15	1918858	BALL JOINT RIGHT	2		
16					
17					
18	1918859	CONNECTION ROD	2		
19					
20	1918857	BALL JOINT LE			
21					
22					
23					
24					
25					
26					
27					
28					
29					
30					
31					
32	1918855	FRONTWHEEL ONE	2		
33	1920632	FRONT BUMPER	1		
34	1920634	WASHER M4 NF E25-513	6		


35	1920633	BOLT M4X20 DIN 7985		
36				
37				
38				
39	1919090	POWER MODULE		
40				
41				
42	1920657	BATTERY CONNECTOR HOLDER	1	
43	1920658	BATTERY CONNECTOR	1	
44				
45				
46				
47				
48				
49				
50				
51				
52				
53				
54				
55				
56				
57	1918861	FRONT CONTACT POINT HOLDER	1	
58				
59	1918860	CONTACT POINT FRONT	4	
60	1910635	BOLT CAP	6	
61				
62				




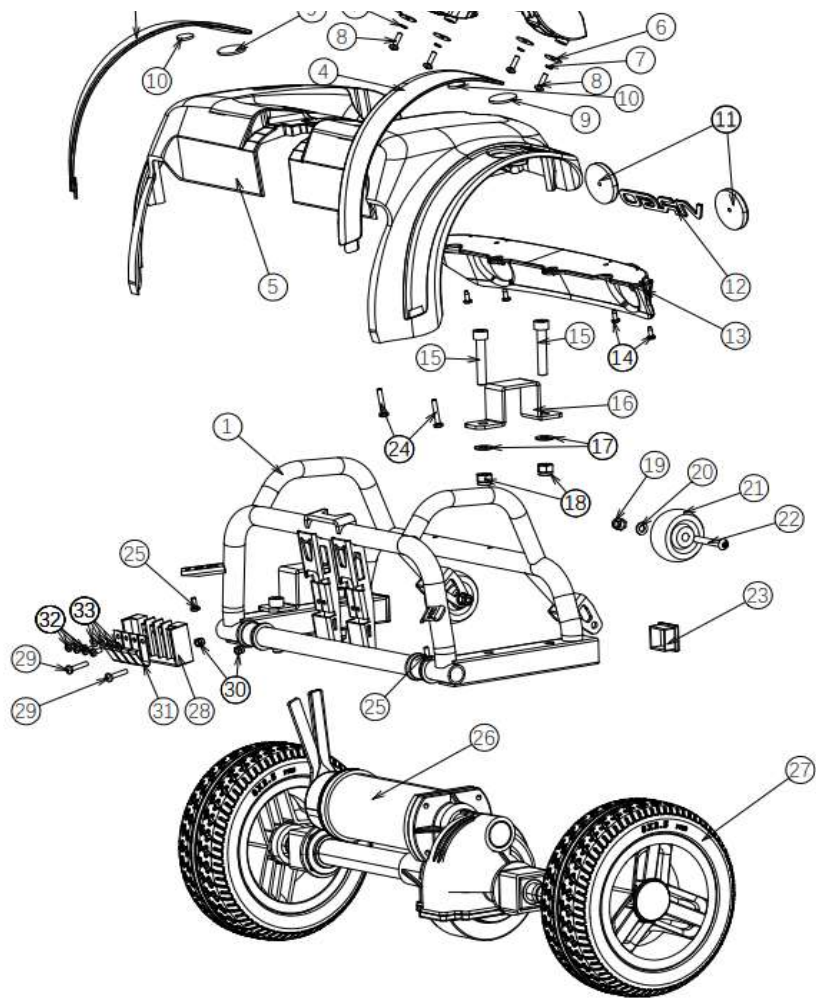

no	part nu	part	QT		part nu	part	QT
1					17		
2	1918886	HANDLE REAR	1		18		


3	1920439	COLOR PANEL SET REAR green		19		
3	1920440	COLOR PANEL SET REAR orange		20		
3	1920441	COLOR PANEL SET REAR bleu		21		
3	1920442	COLOR PANEL SET REAR pink		22		
4				23		
4				24		
4				25		
4				26	1918887	DIFF.+MOTOR+REM 1
				26	1920395	DIFF 1
				26	1920396	MOTOR 1
				26	1920396	MOTOR BRAKE 1
5	1918853	COVER MIDDLE	1	27	1918888	REAR WHEEL 2
6				28	1918889	REAR CONTACT POINT HOLDER 1
7				29		
8				30		
9				31	1918890	CONTACT POINT REAR 4
11	1921454	REAR RED REFLECTOR		32		
12				33		
13	1918882	REAR COVER	1			
14						
15						
16						

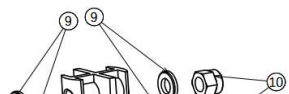

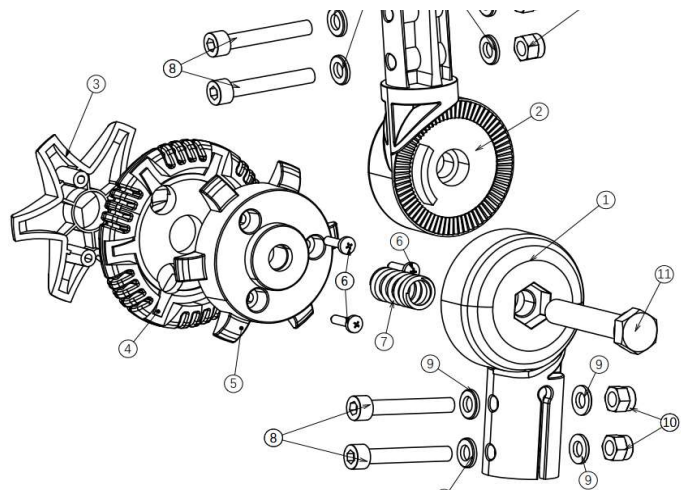


no	part nu	part	QT				
1							
2							
3	1921320	knob angle adjust outside	1				
4	1921319	knob angel adjust middle	1				
5	1921318	knob angle adjust inside	1				
6	1921322	bolt M5 X 16	3				
7	1921321	spring angle adjust knob	1				
8							
9							
10							
11	1921317	bolt steering angle adjust	1				




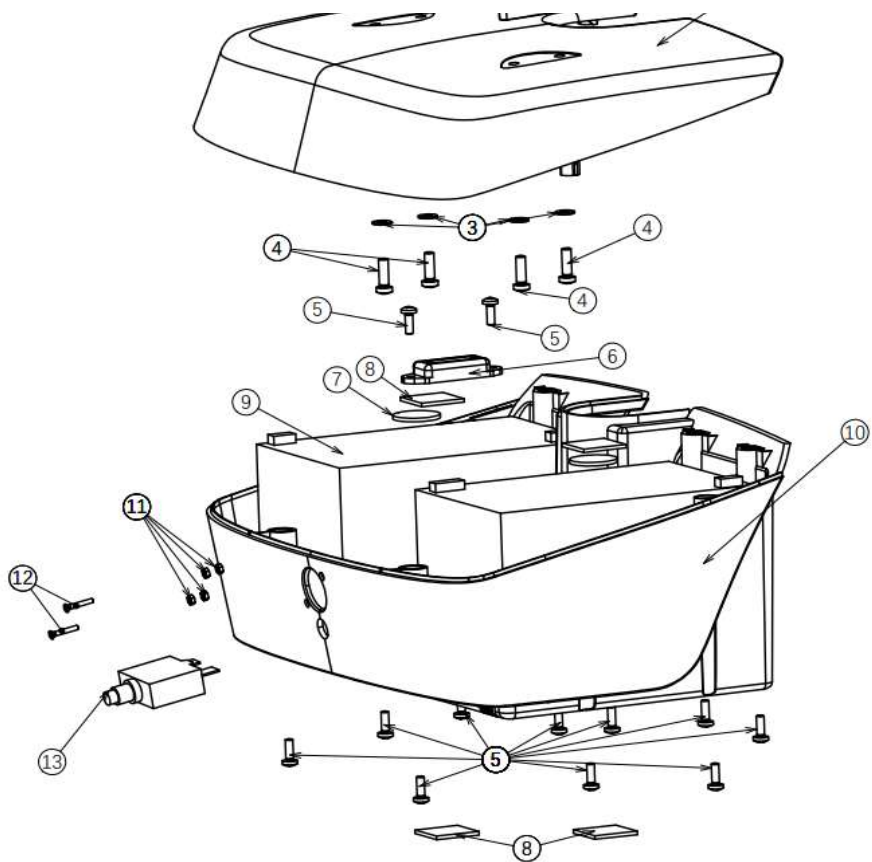

no	part nu	part	QT				
0	1920296	BATTERYBOX COMPLETE	1				
0	1918891	BATTERY BOX WITHOUT BATTERIES	1				
1							


2							
3							
4							
5							
6							
7							
8							
9	1919621	CPX15 -12		2			
10							
11							
12							
13	1920531	MAIN FUSE					
0	1919798	CHARGER		1	not depicted		

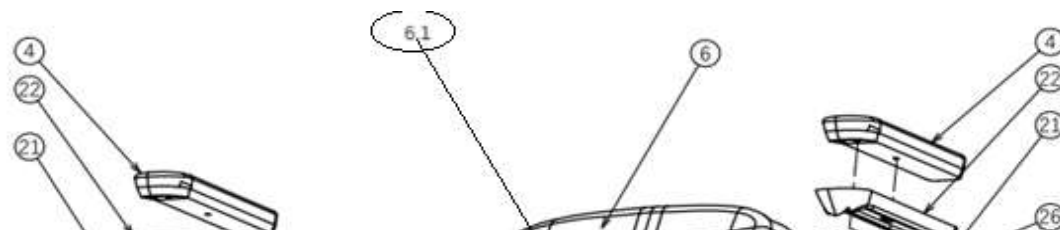


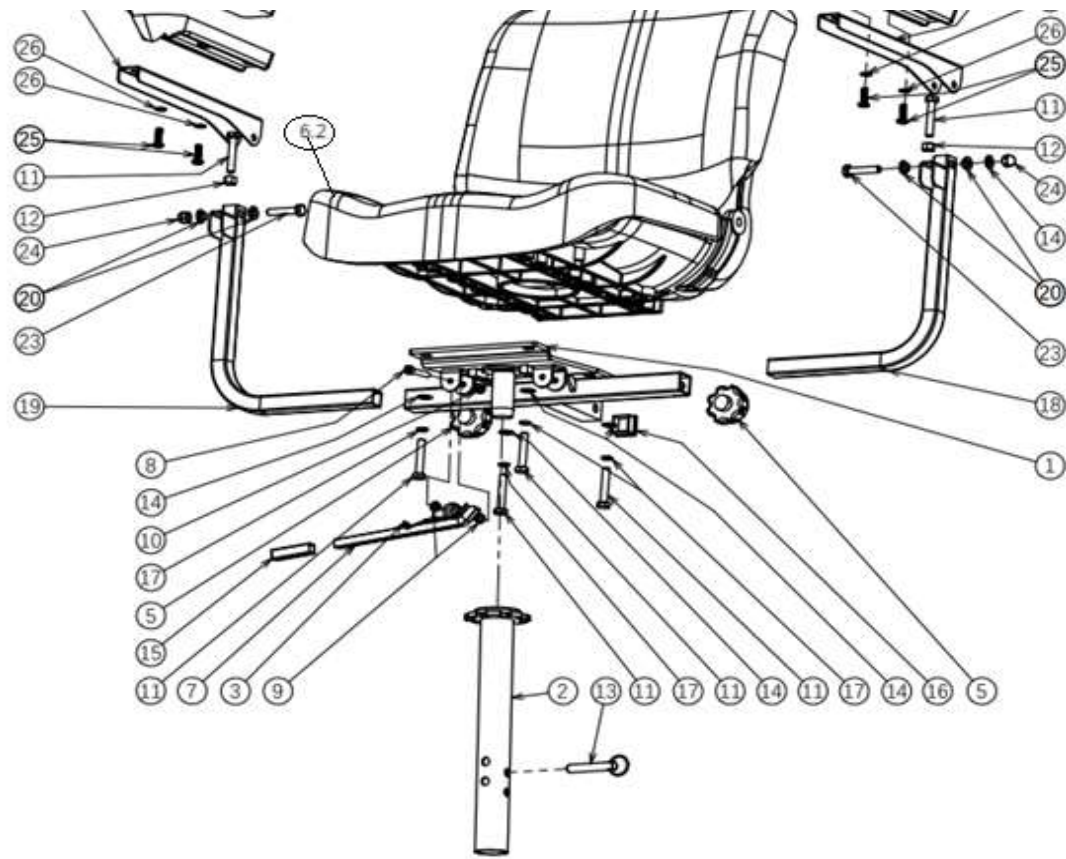

no	part nu	part	QT			
0						
1						
2	1918898	SEAT TUBE	1			
3						
4	1918895	ARM PAD	2			


5	1920266	BOLT ARMREST FIXATION	2			
6	1918899	CHAIR	1			
6.1	1920640	BACK	1			
6.2	1920641	SEAT	1			
7	1920645	SEAT HANDLE	1			
8						
8						
9						
10						
11						
12						
13						
14						
15						
16						
17						
18	1918903	ARMREST LOWER LEFT	1			
19	1918907	ARMREST LOWER RIGHT	1			
20						
21	1918931	ARMRESR UPPER PART	2			
22						
23						
24						
	1920830	ARMREST COMPLETE L	1			
	1920831	ARMREST COMPLETE R	1			
	1921438	SEAT PLATE COMP W/A TUBE	1			

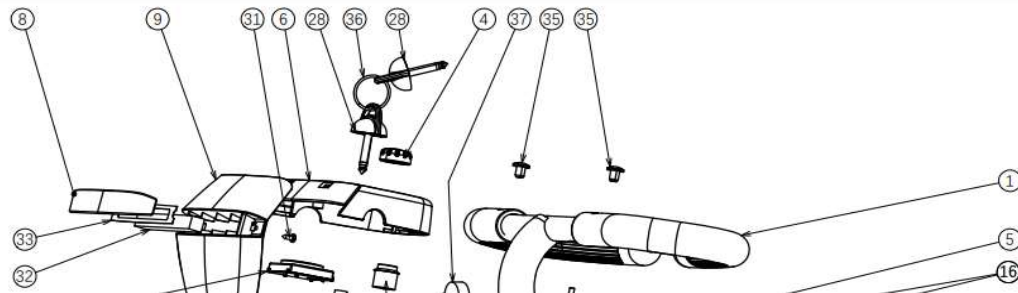


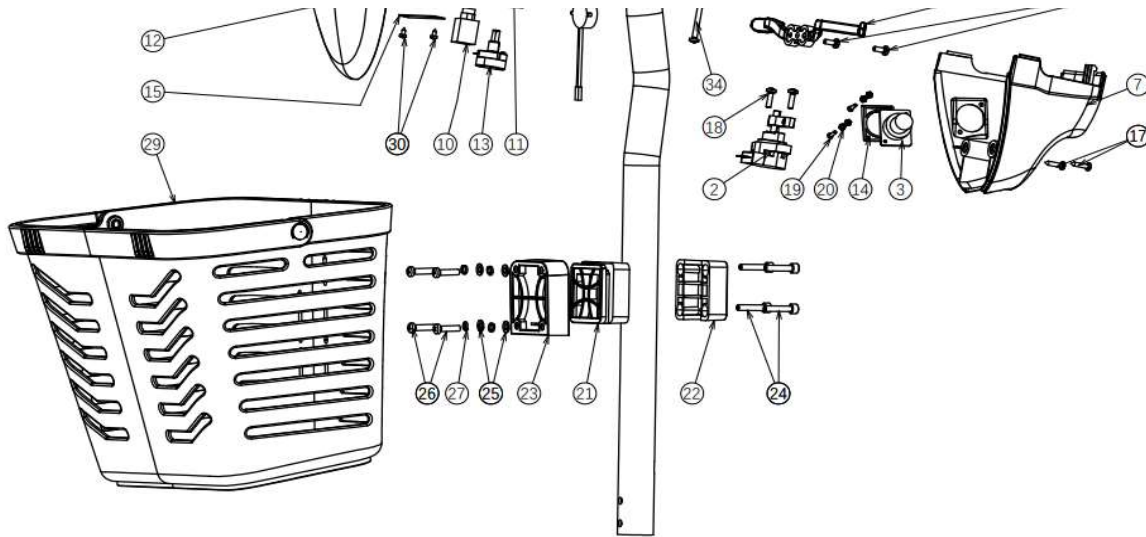

no	part nu	part	QT
1			
2	1918911	SPEED DIAL ASSEMBLY	1
3	1918918	CHARGER SOCKET	1
4			
5	1918910	SPEED LEVER	1
6			
7	1918913	STEERING COVER BACK	1
8	1921463	FRONT LIGHT COVER	1
9	1918914	STEERING COVER FRONT	1
10	1919437	CONTACT IGNITION	1
11	1920423	HORN SWITCH	1
12	1918912	BATTERY INDICATOR	1
13	1918909	POT SPEED	1
14			
15			
16			
17			
18			
19			

20				
21	1920502	ASSEMBLY PLATE BASKET MIDDLE	1	
22	1920510	ASSEMBLY PLATE BASKET REAR	1	
23	1920501	ASSEMBLY PLATE BASKET FRONT	1	
	1920832	BASKET HOLDER COMPLETE	1	
24	1920511	BOUT BZKT M5X25 DIN912	2	
25	1920504	WASHER	4	
26	1920505	BOLT M5X20	4	
27				
28	1918908	IGNITION KEY	1	
29	1920509	BASKET	1	
30				
31				
32	1920629	FRONT LIGHT	1	
33				
34				
35				
	1918977	MIRROR LEFT	1	
	1918978	MIRROR RICHT	1	
37	1920628	HORN	1	

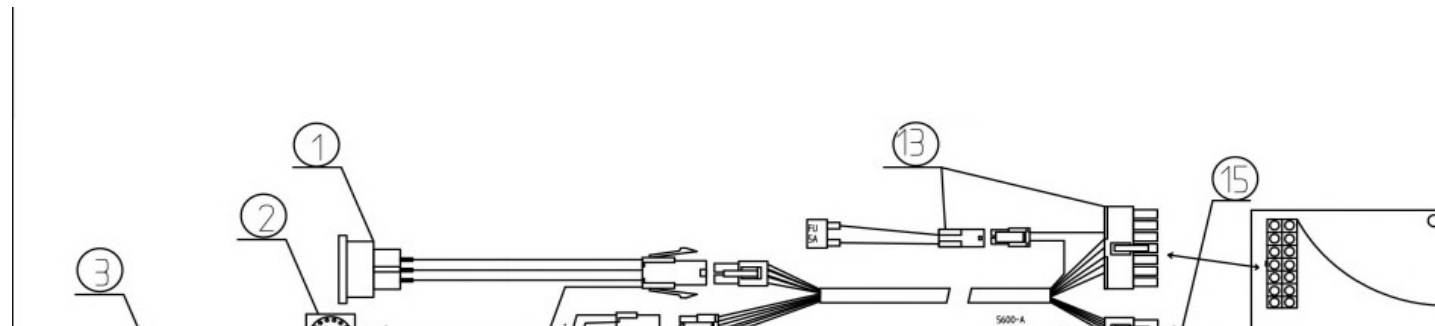

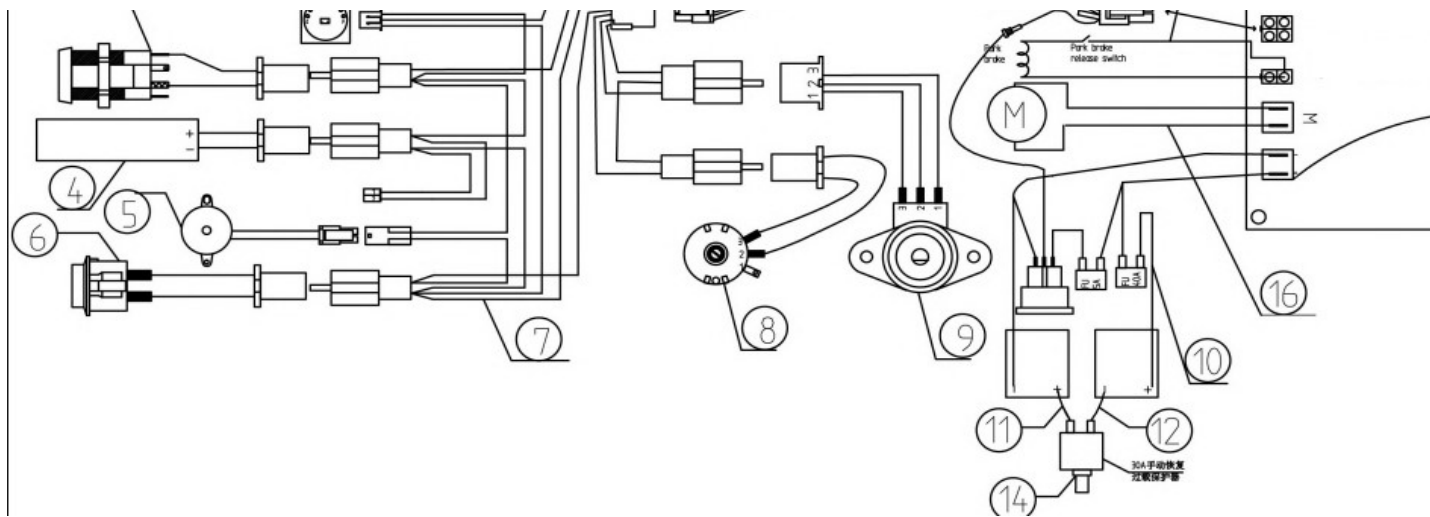

no	part nu	part	QT
1	1918918	CHARGER SOCKET	1
2			
3			
4			
5			
6			
7	1918976	MAIN CABLE	1
8			
9	1918909	POT. SPEED	1
10			
11			
12			
13			
14			
15			
16			